



THERA-PAW, INC

Assistive products for animals with special needs
therapaw.com 908-439-9139 questions@therapaw.com

MEASURING INSTRUCTIONS FOR THERA-PAW BOOT

STEP 1:

Measure the length & width of the paw.

Place a piece of paper on the floor and trace the outline of your dog's paw. Hold your pen at a 90 degree angle (straight down on the paper) to get the most accurate tracing. Use your tracing to measure the **length** and **width** of the paw.

Length: Measure the distance from the tip of the longest nail to the very back of the paw pad.

Width: Measure the width distance, from the right to left side of the paw, at the widest point.



STEP 2:

Measure the paw circumference.

Use a measuring tape to measure the thickest part around the pet's paw.



If you do not have a measuring tape, use a string to wrap around the thickest part of the paw, then take that length of string and measure it with a ruler.

STEP 3: (FOR FRONT PAWS ONLY)

A. Measure the wrist circumference.

For front paw boots, measure around the dog's leg between the paw and the carpal pad (located behind the leg at the level of the dew claw and wrist joint).

B. Measure the wrist height.

Measure the distance from the floor behind the pet's paw to just below the carpal pad (stopper pad located behind the leg at the level of the dew claw and wrist joint).

A



B



STEP 4: (FOR HIND PAWS ONLY)

A. Measure the ankle circumference.

For hind paw boots, measure around the dog's leg half way between the paw and the ankle joint.

B. Measure the ankle height.

Measure the distance from the top of the paw to the ankle joint.

A



B

